



## What's Predictable for?

Predictable is a text-based app for AAC - *Alternative and Augmented Communication*, to [give a voice](#) to people worldwide who live with communication barriers. The eye tracking access method with Hiru enables users to [communicate](#) efficiently using only their gaze.

## How to use Predictable with Eyegaze



Install or update [Predictable 6](#)  
Download it from the App store on your iPad.



[Position and connect Hiru](#) to the iPad  
Place Hiru underneath the iPad in a landscape position. Use the provided cables to connect.



Select Eyegaze as your access method.  
Go to Predictable settings>Access>Eyegaze>Enable eyegaze



Personalize your eyegaze settings.  
Calibrate, enable cursor snap, and adjust cursor visibility autonomously.



Communicate in eyegaze enabled mode.  
Type what you want to say, 2x times faster!



To leave the app, click the red plus sign, then the AssistiveTouch icon.  
AssistiveTouch will appear, and you can leave the app.